

Ovi-Scan 6

Innovative Imaging



Quick Instruction Guide

Operation

Power On

Connect power connector to rear of Console.
Latch Duo-Scan module into console.
Connect Probe to front of module.
Press and hold Freeze/Power button until unit beeps.

Range Control Via Tally

Press Tally buttons 1&2 to increase range.
Press Tally buttons 3&4 to decrease range.


Power Off

Press and hold Freeze/Power button until Console switches off.
Note: Probe will rotate until button is released.


The Buttons




Scrolls through the Main Menus

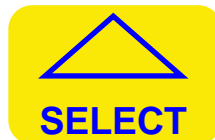

Scrolls through sub-menus or activates option shown in Menu.

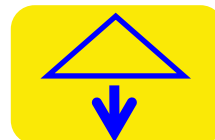

Scrolls through sub-menus or activates option shown in Menu.

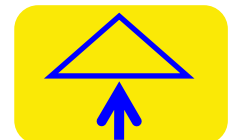

Scrolls through sub-menus or activates option shown in Menu.



Short Press Freezes Image.
Long Press Switches On & Off.

BCF Technology Ltd
3 Tailend Court
Starlaw Road
Livingston EH54 8TE

Email: info@bcftechnology.com
Web: www.bcftechnology.com
Tel: +44(0)1506 460023
Fax: +44 (0)1506 460045



The Menus

TALLY Press Setting to advance to next menu	RUN/TOT Press Select to switch between Running & Total Tallys	ADD+CLEAR Press ↓ to add the Running tally to the Total and clear the Running tally.	NEG TALLY Press ↑ to activate negative tally then press the appropriate button on the tally controller.
DISPLAY Press Setting to advance to next menu	BRIGHT 12 Press Select to switch between Brightness & Contrast	↓ Press ↓ to decrease Brightness or Contrast	↑ Press ↑ to Increase Brightness or Contrast
CONTROL Press Setting to advance to next menu	RANGE Press Select to switch between Range, Gain, Power & Frequency	↓ Press ↓ to decrease Range, Gain, Power or Frequency	↑ Press ↑ to Increase Range, Gain, Power or Frequency
IMAGE Press Setting to advance to next menu	INVERT Press Select to Invert Image	REVERSE Press ↓ to Reverse Scan Direction	FREEZE Press ↑ to freeze Scan image.
FOETAL AGE Press Setting to advance to next menu	OFF Press Select to switch between Off, Cattle and Sheep	CRL Press ↓ to cycle through the available circles	ON Press ↑ to switch selected circle on or off
DRAFTING Press Setting to advance to next menu	TALLY 0 Press Select to switch through the Tally values 0-4 & Late	DELAY Press ↓ to switch between Delay or Gate 1-4.	100 Press ↑ to select Gate delay or switching
RFID Press Setting to advance to next menu	No Function	FLASH OFF Switches "Flash on Tally Press" on or off	TALLY CLEAR Press ↑ to Clear Total Tally
MODES Press Setting to return to Tally menu	OVI-STORE Press Select to store current Ovi-Scan configuration	DUO-STORE1 Press ↓ to store current configuration as Duo-Scan mode.	RF CHAN 3 Press ↑ to change the RF Channel